

CLEARMASK

Single Colleague Use

Colleague Restrictions:

1. For use when interacting with patients who are hearing impaired or need to read lips to communicate;
2. For colleagues who can't wear a surgical mask for any medical reason approved by colleague safety.

Putting on ClearMask

- Clean hands with soap and water or hand sanitizer.
- Remove a single ClearMask from packaging, and peel off the **protective layer** on the outward-facing side of the mask. Inspect for any damages, rips, holes or contamination. Do not use product if any damages or contamination are found.
- Position the mask in your hand with the foam facing inward and the nose cutout on top. Bring the mask toward your face, such that your nose fits into the nose cutout and your chin rests on the bottom foam section.
- With your other hand, take both the top and bottom straps and place them over your head. The bottom strap sits around the base of the head. The top strap rests higher at the back of the head, above the ears. *Do not crisscross straps.*
- Use the EZ-adjuster in the back to adjust the mask by pushing inward to tighten or pulling outward to loosen the straps.
- Ensure proper fit and direct contact between the face and the foam without gaps

Removing ClearMask

- Clean hands with soap and water or hand sanitizer.
- Remove the ClearMask by pulling the straps up and over the head, loosening the adjuster piece if needed.
- Slowly pull the mask away from the face. Avoid touching the front of the mask or any liquid spalltter on the mask.
- Clean the ClearMask with PDI wipes or Hypercide (follow contact time and cleaning process).
- Place in paper bag and continue to reuse until soiled or damaged. Discard in regular trash.